


IMPRESSION REQUIREMENTS FOR SOMNOMED DEVICES



- ✓ Full Upper & Lower Arch
- ✓ PVS or Silicone Impressions both heavy & light body materials
- ✓ Alginate may be used if the models are poured immediately in more than Type 3 Stone.
- ✓ It is not recommended to use a polyether material such as Impregum; the resulting impression is too stiff and prone to breaking plaster models.

QUALITY

- No tooth distortions or drag
- No large holes or bubbles (>0.5mm diameter)
- Impression of all teeth including some portion of last molar
- Soft tissue coverage 5mm beyond the gingival margin, buccal of the last molar
- Full sulcus extension

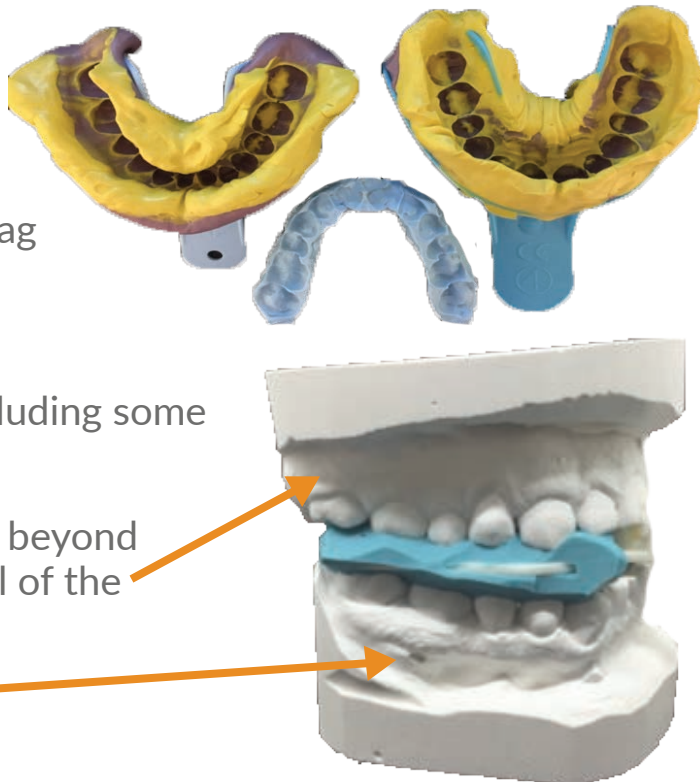

OPTIONAL FEATURES:

- **Request to use Acculiner Articulator:** Coverage of the hamular notches and incisive papilla
- **Edentulous:** Coverage of the entire palate
- **DentiTrac Micro-recorder:** Extension 7mm buccal of canine & first molar on left hand side